


TriCELL

Preparation Manual

TriCeLL Preparation Manual

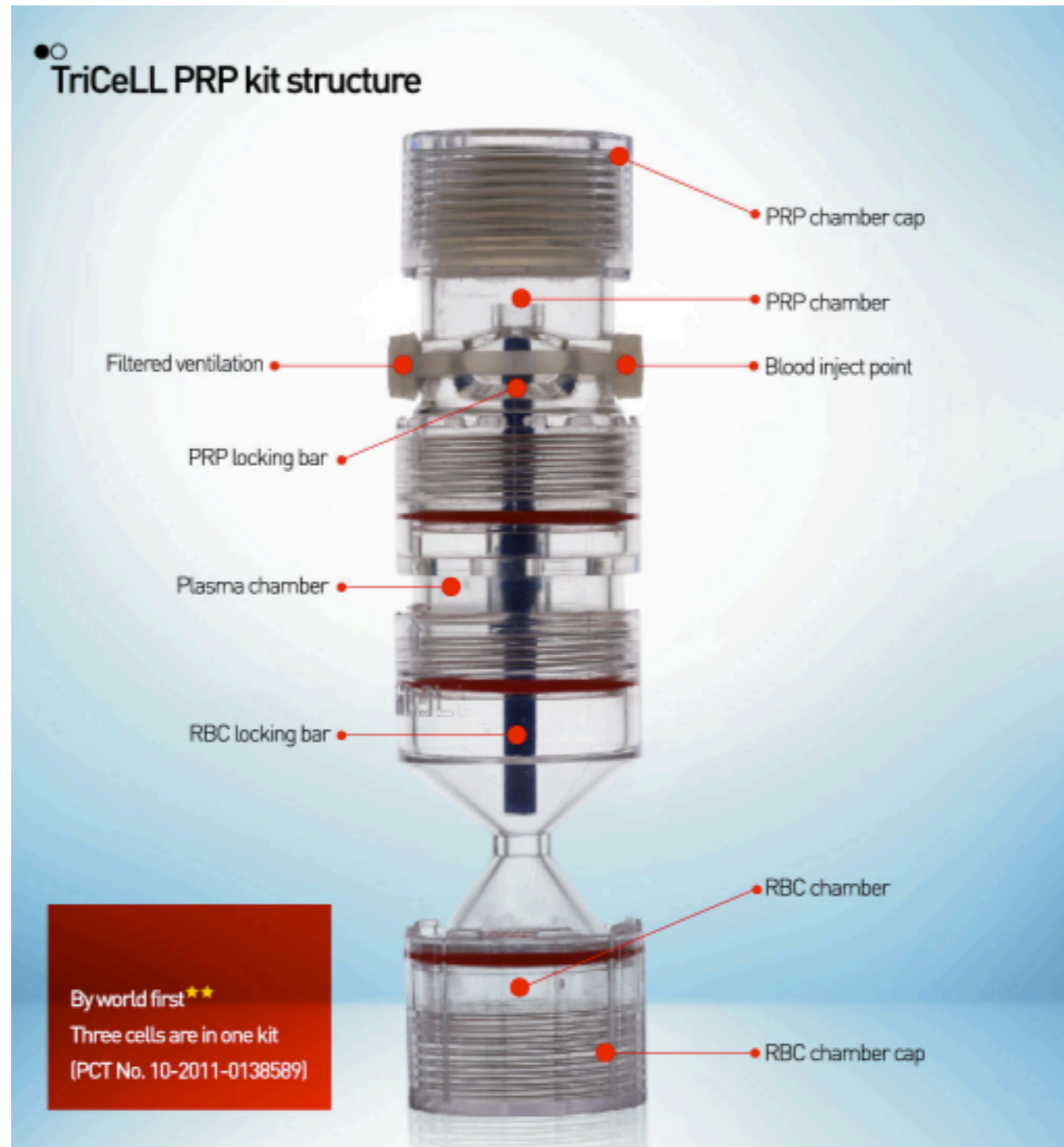

TriCeLL Preparation Manual

Preparation:

30cc syringe 1ea (19~21g) for Blood withdrawal

5cc syringe 1ea (21~23g) for extracting PRP


1cc syringe 5ea (21g) optional for PPP filler



Before blood withdrawal, add ACD / A in the ratio of 1: 10 in a syringe.

**In TriCeLL, Male 30cc blood and add 3cc ACD-A
Female 32cc blood and add 3cc ACD-A**

TriCeLL Preparation Manual

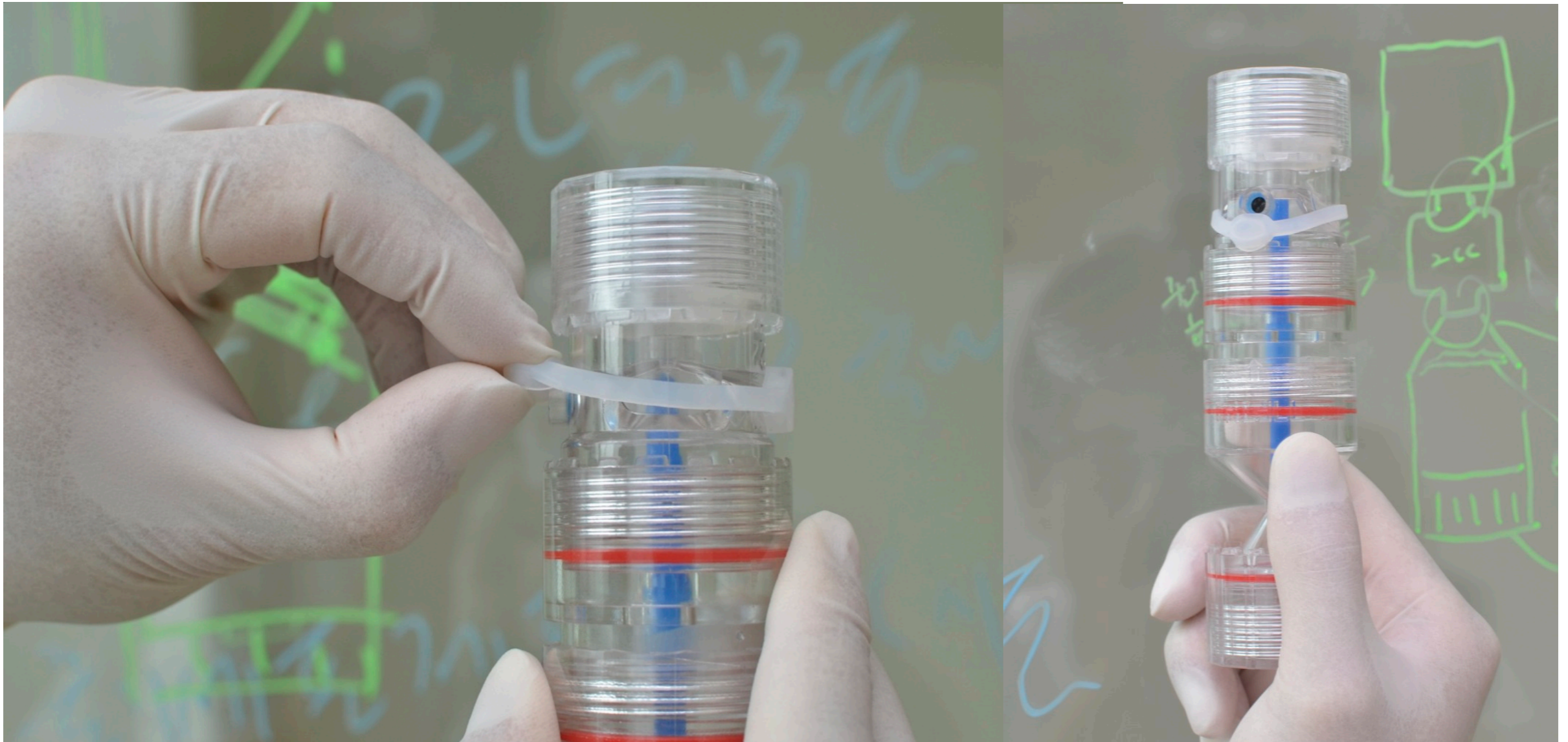

Open the filtered ventilation cap before Blood injection

TriCeLL Preparation Manual

**Insert the needle to the
injection point**

*** Should give 30° angulation to
prevent blood leakage through
filtered ventilation**



TriCeLL Preparation Manual



Make the same weight $\pm 1g$

TriCeLL Preparation Manual



3200rpm / 5 min

TriCeLL Preparation Manual



After 1st centrifuge

TriCeLL Preparation Manual



Turn the RBC chamber cap to move buffycoat to the plasma chamber

TriCeLL Preparation Manual



**Hold upper body of TriCeLL
and Turn the down part of
TriCeLL to isolate RBC with
interlocking bar**

*** Lock it tight as strong as you can**

TriCeLL Preparation Manual



**Mix Plasma with
buffycoat**




**Plasma + buffycoat
Make it **Pink** !!**

TriCeLL Preparation Manual



Flip it over
Before 2nd Centrifuge,
adjust prp density amount
From 1cc to 4cc

TriCeLL Preparation Manual

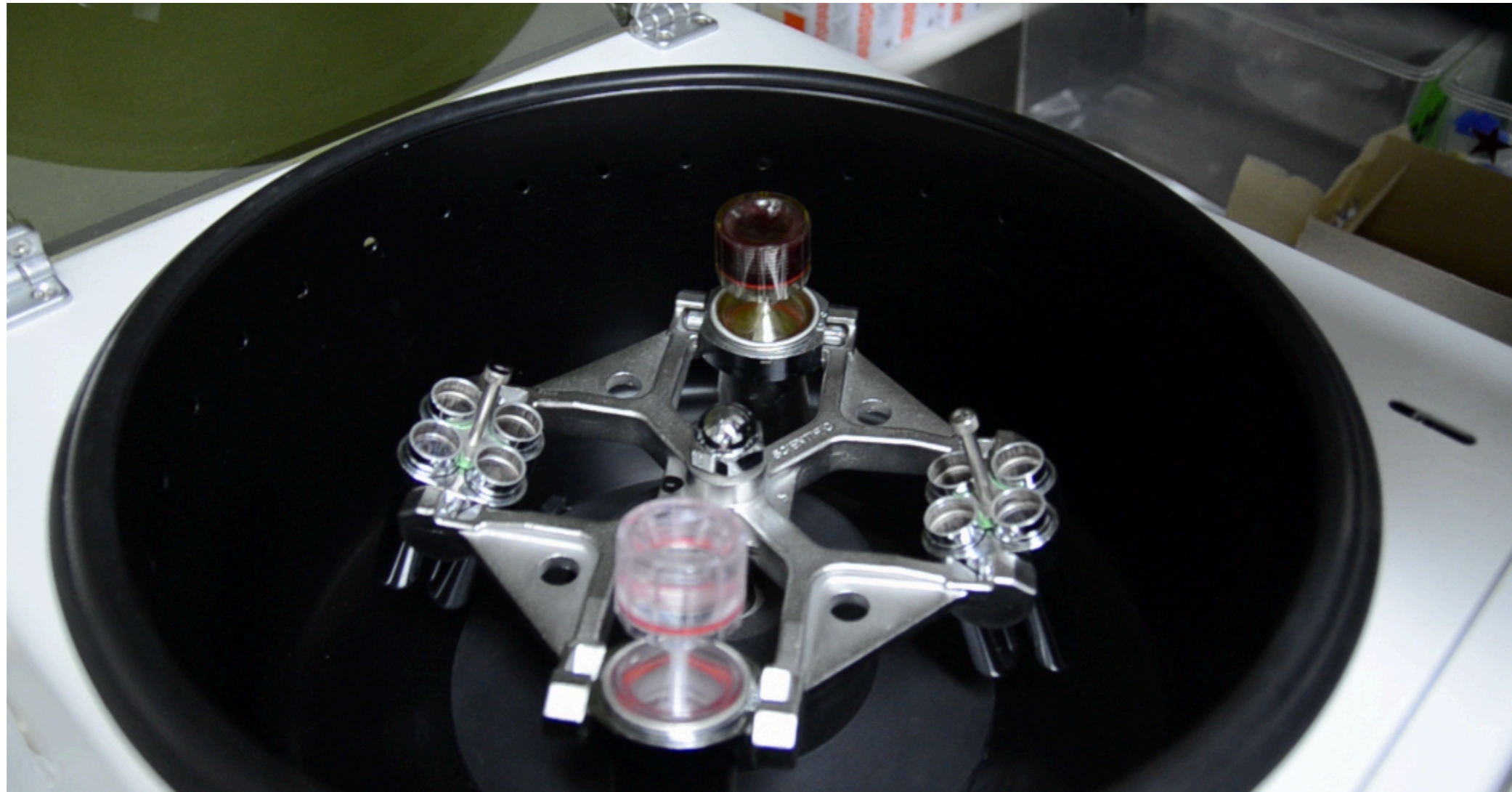

2nd centrifuge
3200rpm / 3min


TriCeLL Preparation Manual



**Hold the TriCeLL body and
Turn the blood injection
point to isolate PPP and PRP**

(Do not turn PRP cap!!)

TriCeLL Preparation Manual

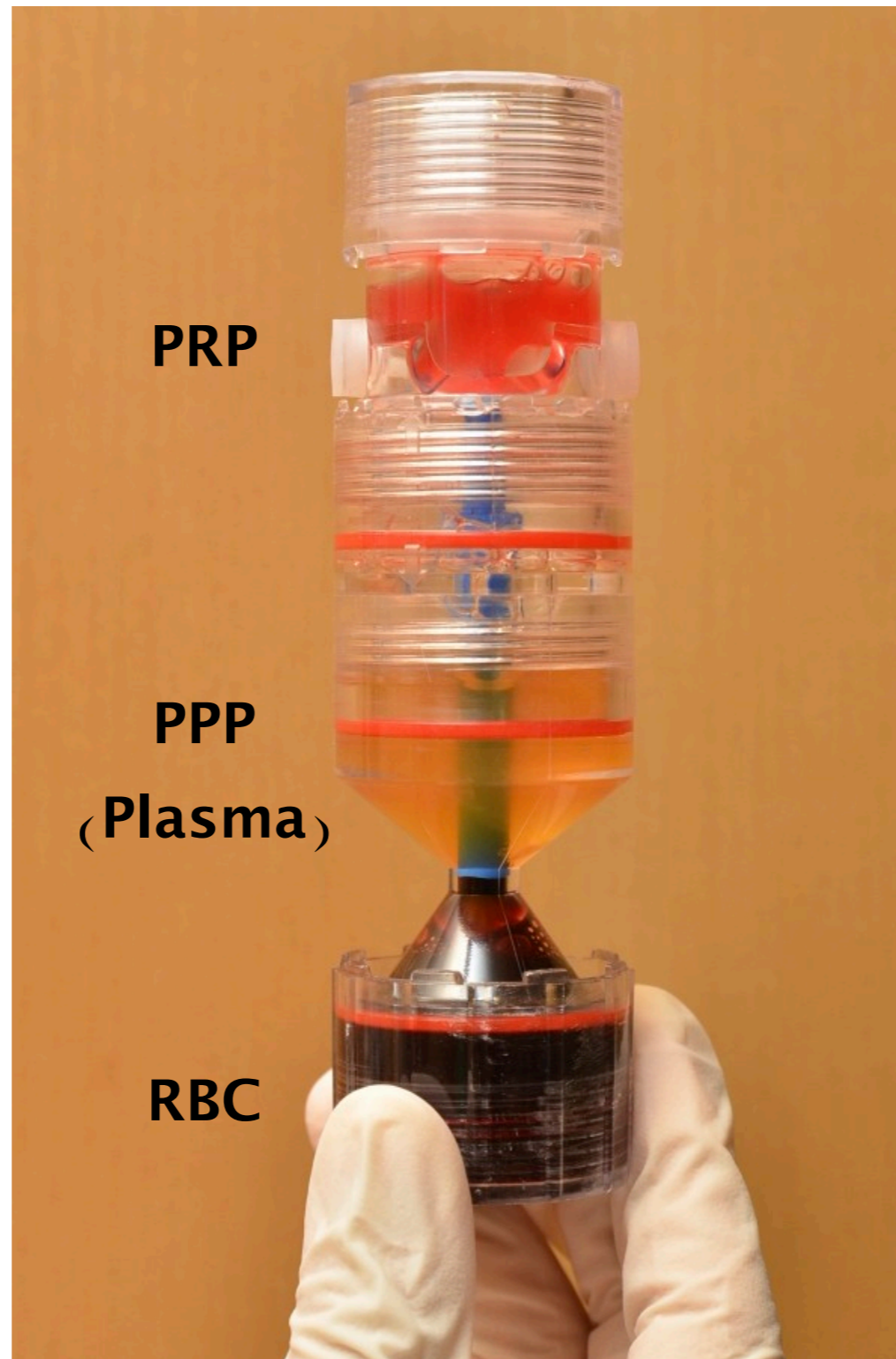

TriCeLL Preparation Manual





Feel of the sealing

TriCeLL Preparation Manual



Put the syringe through prp cap and harvest PRP

TriCeLL Treatment for Pain management

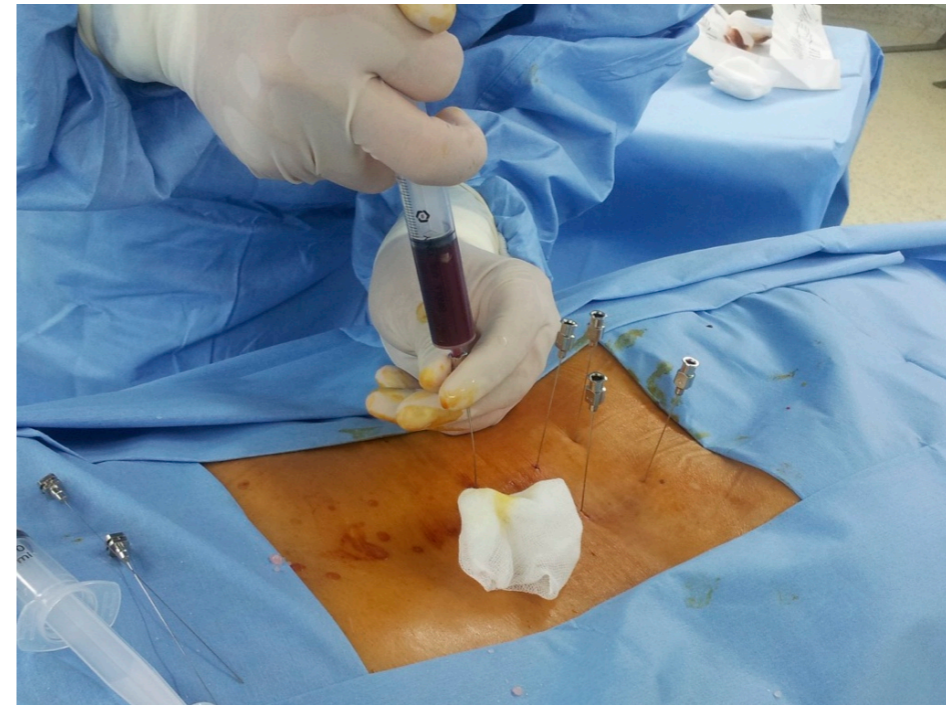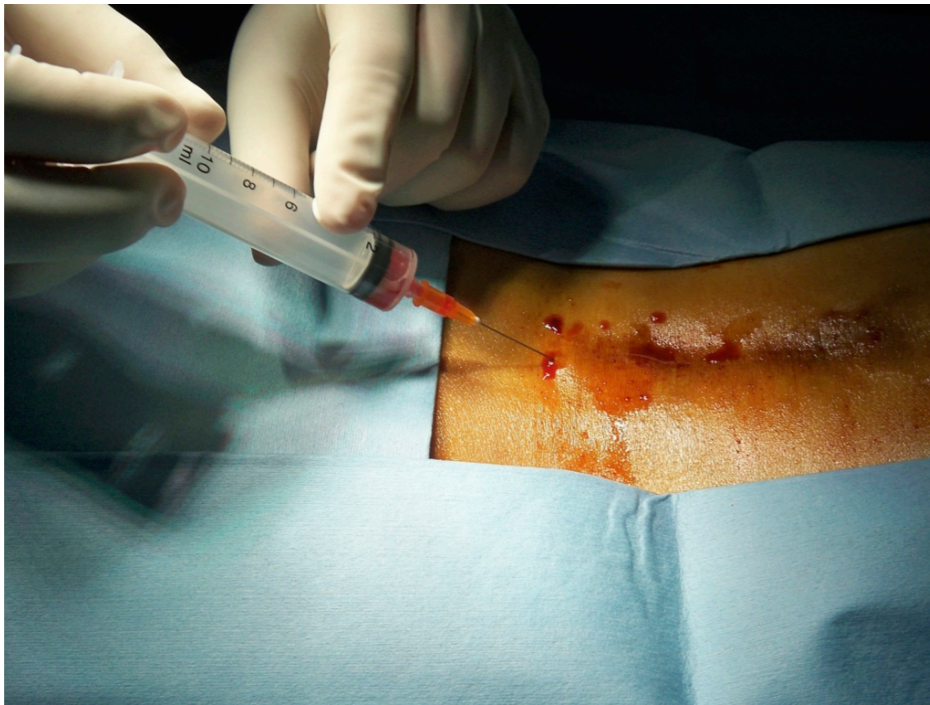

TriCeLL Treatment for Anti-aging



Thank - you